

Breas Apto® F 01

Fitting instructions



1. Remove both lower headgear clips prior to fitting.



2. Place mask on to the face, the top of the mask should align with the top of the nose bridge. Pull the headgear over head.



3. Ensure the lower strap sits below the ears and secure both lower headgear clips.



4. Adjust both upper headgear straps evenly, ensure the mask feels stable.



5. Adjust lower headgear straps evenly.



6. Connect the elbow with the tubing attached and start your device.



7. For long hair, pull the hair between the crown and the base of the headgear.



8. All set!

References: Apto® F 01 Instructions for Use; Breas Data on File.

Breas, the Breas logo, Apto, and Breas Apto are trademarks of Breas Medical AB. © 2026 Breas Medical AB - All rights reserved. Breas Medical AB reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your Breas representative for the most current information.

Issued: April 2026. MAR-008193 Rev-1.



Breas Medical AB
Företagsvägen 1, SE-435 33
Mölnlycke, Sweden
+46 31 86 88 00

Be more™



For more
information
please visit
breas.com