



Connecting iLink to a Breas Ventilator or Device

Start

Patient known to have used Ventilator or expect data to be available to view?

Review Data Viewer, Last Sync Time, Patient Status, Set-Up Date & if assigned a Ventilator in Pt. Record?

Any reported/visible Ventilator, cable(s) or iLink damage?


Query if Ventilator is connected to the iLink and both are powered on?

Clarify LED Status on front of iLink e.g. Green, Blinking or Solid Red?

Reboot iLink
(Press Power button or unplug and reconnect power.
The LED will start RED before flashes RED/GREEN to ...

Passed Self-Test
Solid Green LED

Wait 15 min and check Data Viewer & Last Sync Time in PATIENT Record for data in EveryWare.

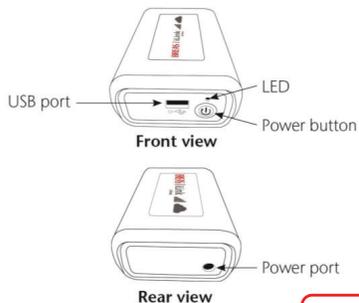

Failed Self-Test

Solid Red LED

In Order of Priority

Temperature Sensor
1 Red Flash
followed by 2 sec. pause

Modem Connection
3 Red Flashes
(at 0.25 sec. intervals)
followed by 2 sec. pause

Device Connection
2 Red Flashes
(at 0.25 sec. intervals)
followed by 2 sec. pause

Blinking Red LED

After Blinking for a specific sequence for 20 sec. the Red LED will stay in steady dim state.

Try moving iLink to find a stronger signal* → Reboot iLink to test. *If unresolved*

Query if Ventilator is connected to the iLink and both are powered on? → Reboot iLink to test. *If unresolved*

If unresolved

Query/update Ventilator serial number(s) assigned in EveryWare? → Check if Firmware versions of Ventilator(s) are compatible? *If unresolved*

Contact Customer Support



Connecting iLink to a Breas Ventilator or Device

1. Connect the iLink to the Breas Ventilator or Device by plugging the USB cable into its USB port and to the front of the iLink.
2. Plug the power adapter supplied with the iLink into the Power port on the rear of the iLink and into the wall receptacle, ensuring switch is on.
3. If needed, press the Power button on the iLink. Make sure the Ventilator or Device is also powered on.

For Technical Support:

Email: everywaretechnicalsupport@breas.com

<https://breas.us/everyware-support/>

Customer Support Line: 855-436-8724

Monday through Friday 8am to 5pm EST